BRANCARD PIGUILLEM**PIGUILLEM STRETCHER**

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Originally made for mountains rescue operations, the "Piguillem" stretcher is now used for multi-purposes as:

- Rescue in narrow area, difficult access (building fronts, spiral staircase)
- Ambulance works, mining and carry rescue
- Helicopter rescue works

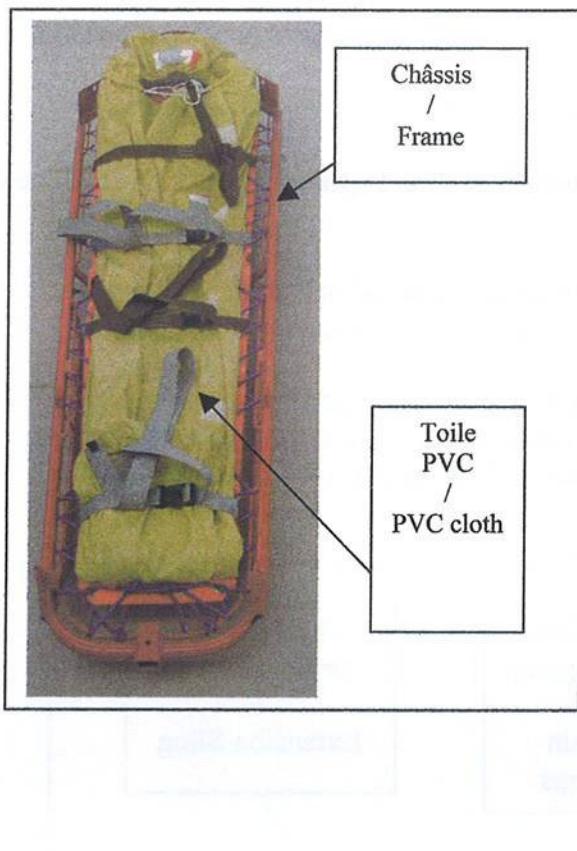
The "Piguillem" stretcher is foldable for backpacking to a rescue site, can be hand carried or shoulder carried using removable Y-shaped extension pieces.

The integrated lifting points and the supplied slings allow helicopter rescue.

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
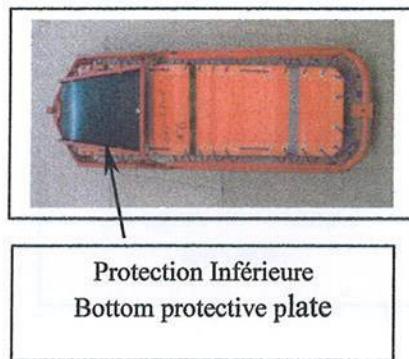
A steel pipe frame with Epoxy coating, equipped with a bottom protective plate and lifting points adapted to different lifting solutions. A stainless steel model is available with painting if required.

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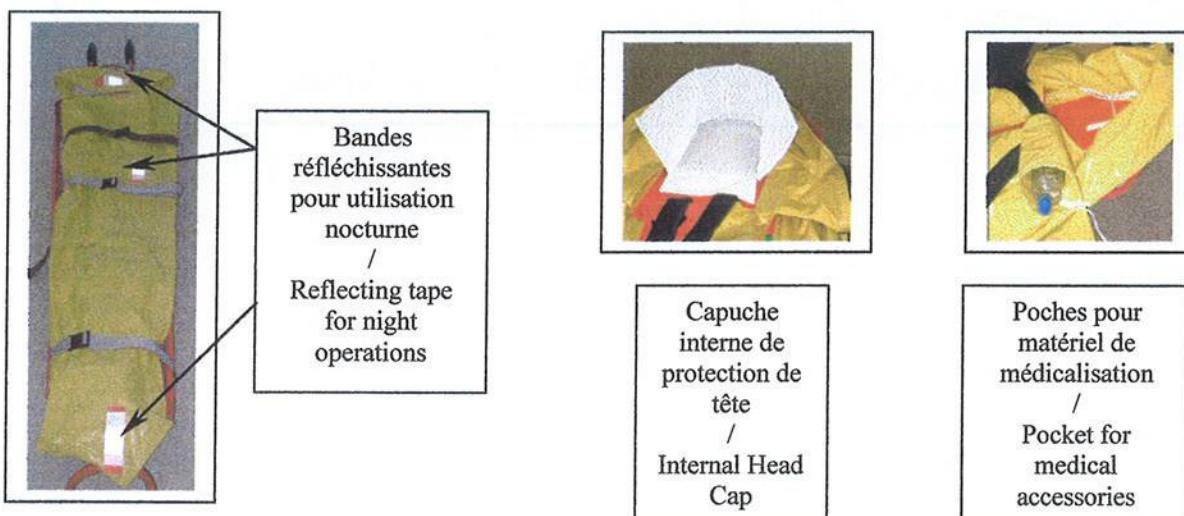The cleaning of the cloth is possible with water, the drying is also quick.

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
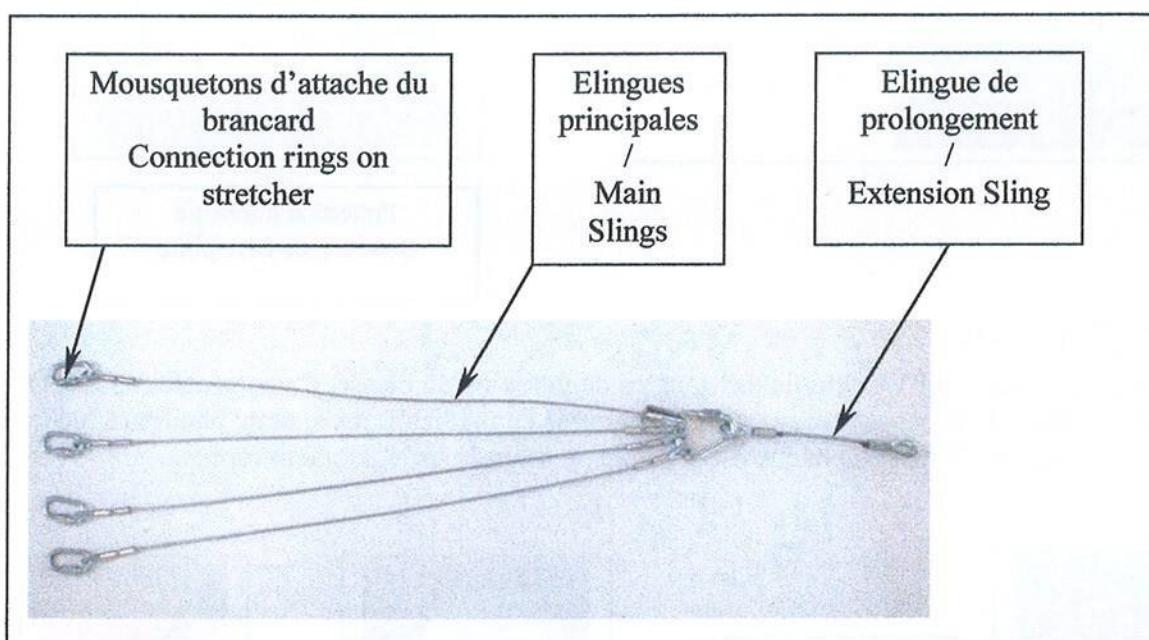
One set of slings for horizontal or canted lifting. This set is composed of 4 main slings to be connected directly on the frame lifting points and an extension sling assigned to make the boarding easier.

The lifting can be realised horizontal or canted according to rescue operation requirements.

For lifting operation from helicopter, we recommend the canted position for the stretcher. This position reduce the stretcher gyration due to the stream of helicopter rotor.

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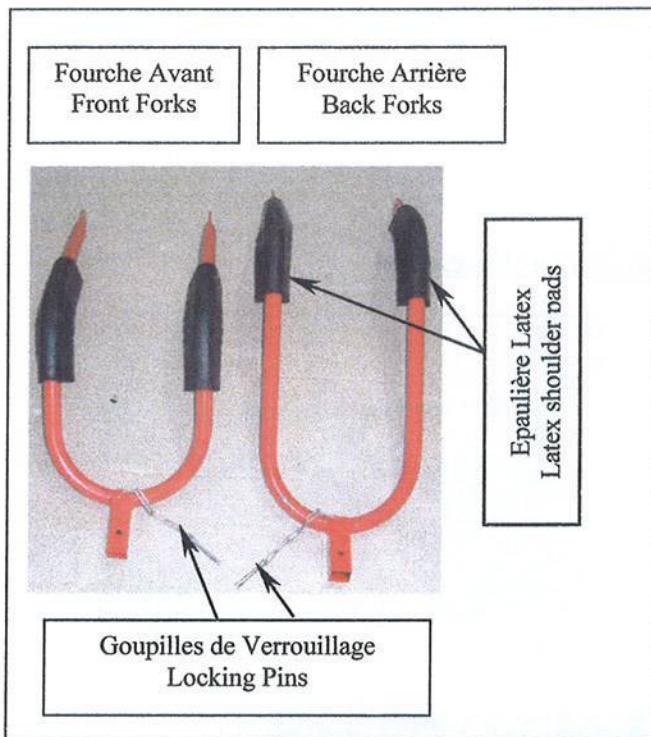
A removable front and a long back forks allow the carriage of the stretcher by two persons on foot path. The forks provide more convenience during handling operations. These forks are equipped with latex shoulder pads for carriage comfort . The carriage of the stretcher by four persons is possible by the lateral handfuls.

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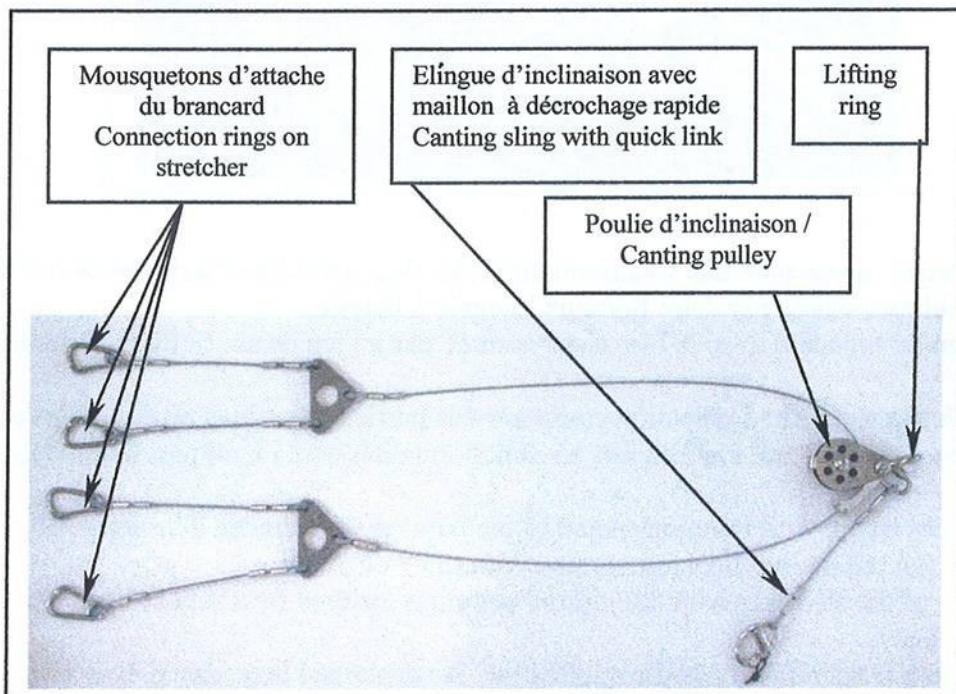
This set of slings allow only canted lifting operations and give the possibility for the final stage of lifting operation, to modify the stretcher position from oblique to horizontal. The only action required to do such is to open the quick link. After the stretcher canting is managed by the operator.

This function is very helpful for the boarding of the stretcher in a rescue helicopter. The canted position reduces the stretcher gyration due to the stream of helicopter rotor.

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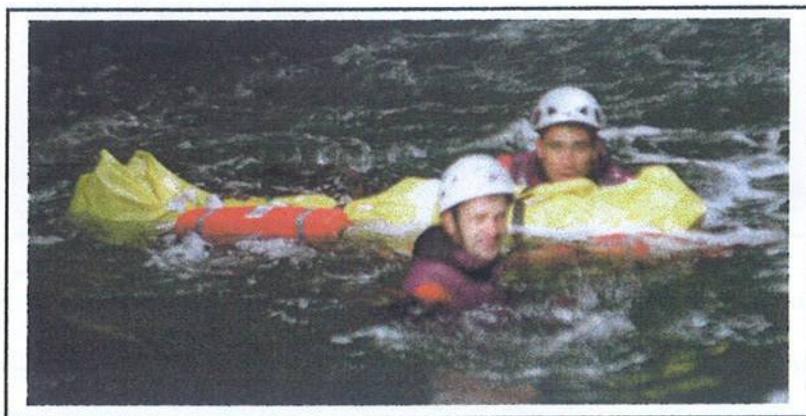Stainless Steel frame with functions equal to the painted steel model. The self-floating of the stretcher is ensured by two floats integrated with the PVC cloth.

The floating of the stretcher with the injured person is ensured by a set of six external and removable floats.

The PVC cloth is specific to aquatic application. As described here above, two floats are integrated; some areas are made of mesh to evacuate the water. In the same way some areas are specifically designed to facilitate the water evacuation. The other functions are similar to standard model.

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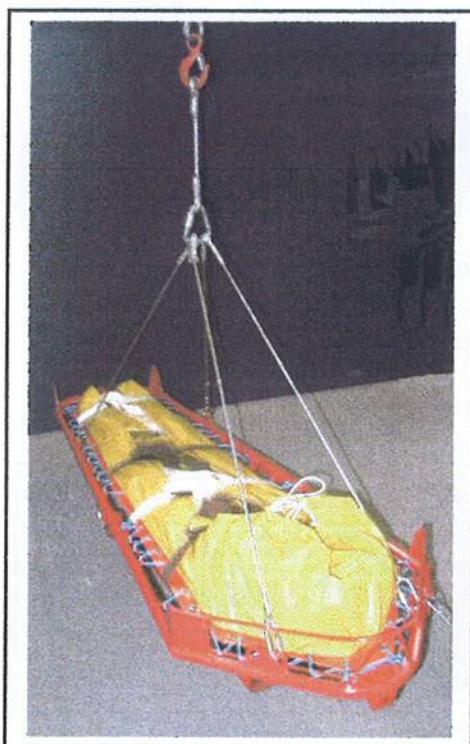
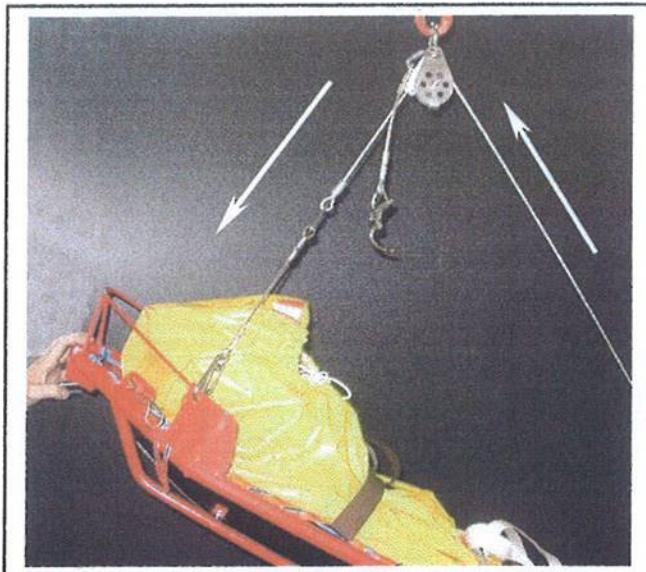
Canted lifting position. The canting sling is connected to the stretcher

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After opening of the quick link, the operator can manage the stretcher canting for the boarding.

3.6 Portage-Stockage / Carrying-Storage

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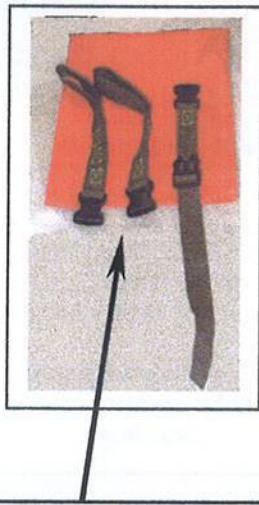
In folded position, the use of the shoulder straps connected to the removable lower straps allow a carriage as a back pack

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The storage of the removable straps bag in the medical pockets is possible

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
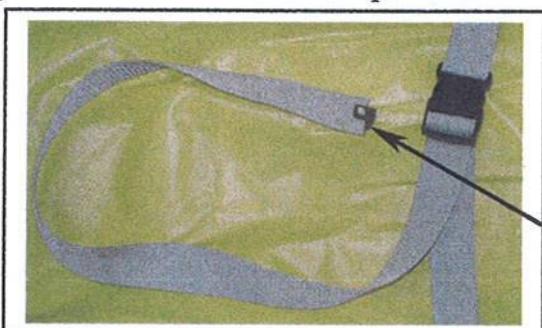
Verrou d'extrémité de sangle.
Webbing extremity stopper

The holding of the injured person is ensured with two sets of straps. The first set is inside the cloth; the second ensure the closing of the cloth on the injured person and a second holding.


All the webbing's are hot cut to avoid any fringe of the webbing thread.

The webbing's, which should be connected together, have the same colours. Plastic quick buckles are used for all the webbing's connections. The resistance of these buckles is 100 daN for the large model. A replacement kit is available for the buckles.

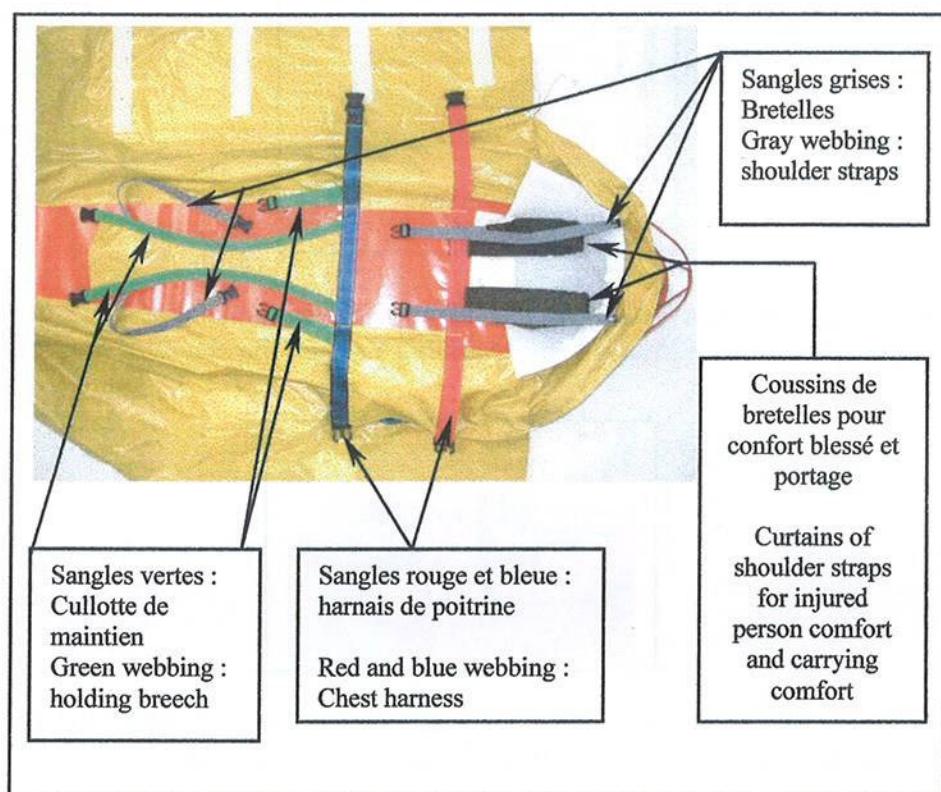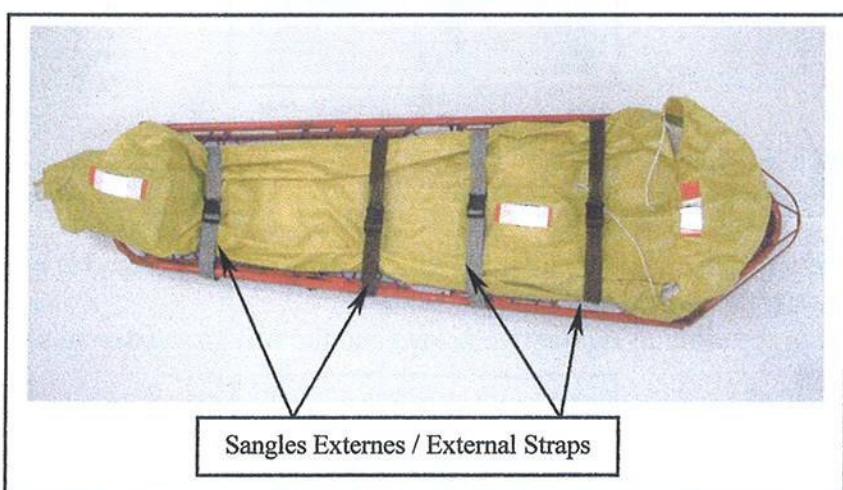The use of webbing extremity stoppers avoids any loss of plastic buckles.

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