



FRANCO GARDA STRETCHER
SINGLE WHEEL
User Manual

MU PFFG028 ed.02-2021



SINGLE WHEEL
USER MANUAL





FRANCO GARDA STRETCHER SINGLE WHEEL

User Manual

Warning:

This user manual only applies to the single wheel that can be fitted to the Franco Garda stretcher.

For information on the use of the Franco Garda stretcher, refer to its instructions for use.

When using the Franco Garda stretcher and its accessories, the safety of rescuers and patients depends on the stretcher being used properly.

All persons using it must have read the user manual and have carried out practical exercises before setting off for an operation. We recommend using a dummy for these exercises.

To ensure the proper functioning of the Franco Garda stretcher and its accessories, carefully observe the recommendations in the "Maintenance" paragraph.

Very important instruction:

- Never set off for an operation without first checking the condition of the single-wheel and the stretcher,
- In the event of a problem seek advice from a TSL RESCUE technician or return the part concerned,





FRANCO GARDA STRETCHER SINGLE WHEEL


User Manual

1/ Presentation:

The single wheel is for use with the PFFG031 or PFFG045 carrying systems. It fits onto the carrying bars and helps to reduce the strain on the stretcher bearers. The single wheel consists of a tubular aluminium frame, on which four mounting brackets slide. These brackets are fitted with hooks for attaching the single-wheel to the stretcher.

The wheel has a large diameter to facilitate transport even on very rough surfaces. This wheel is fixed on a swinging arm to improve patient comfort.

Once the mounting brackets are installed on the chassis, the wheel fits under the stretcher carrying the patient.

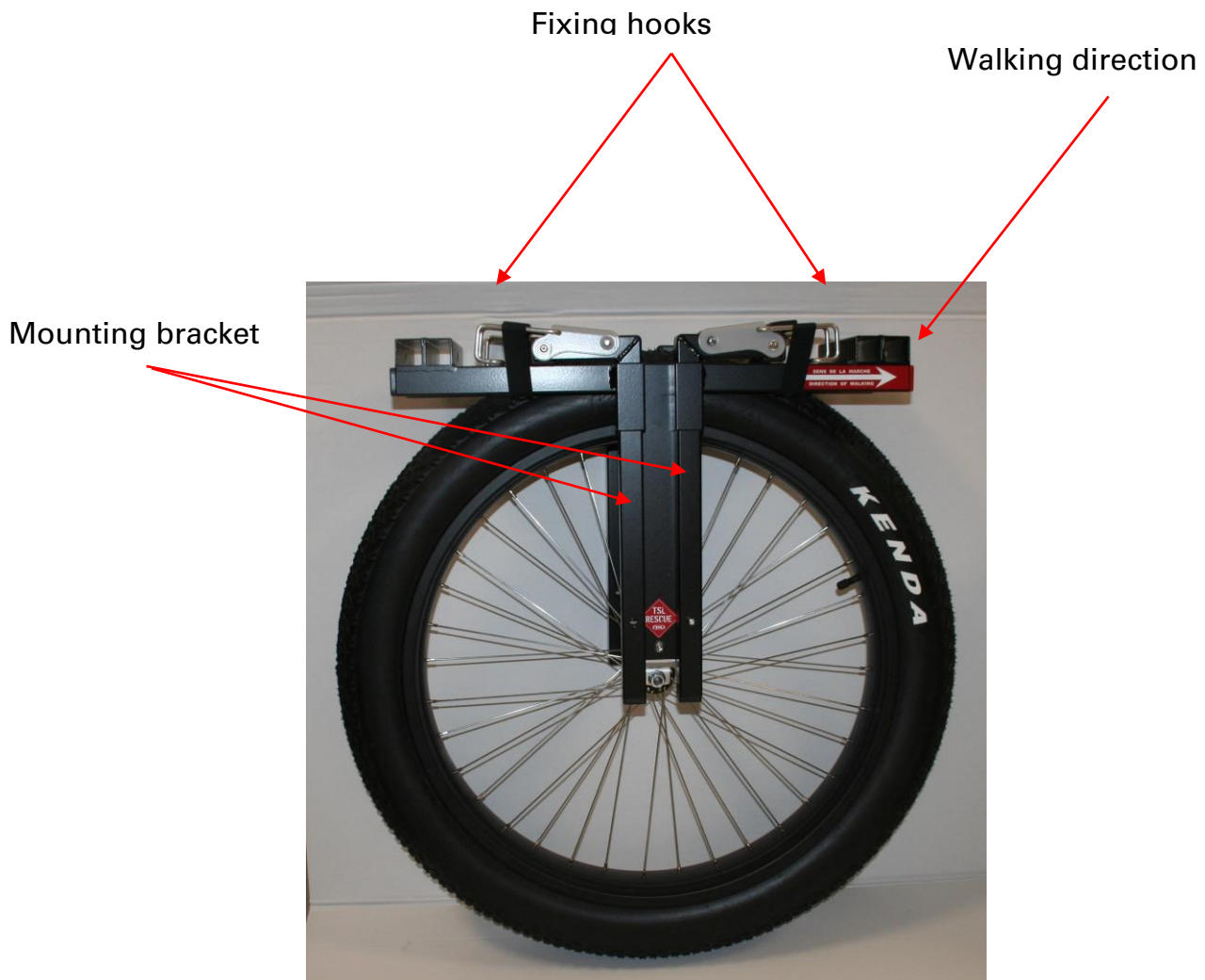




FRANCO GARDA STRETCHER SINGLE WHEEL


User Manual

2/ Fitting the single wheel to the stretcher:

The carrying bars must be fitted to the stretcher before the single wheel.

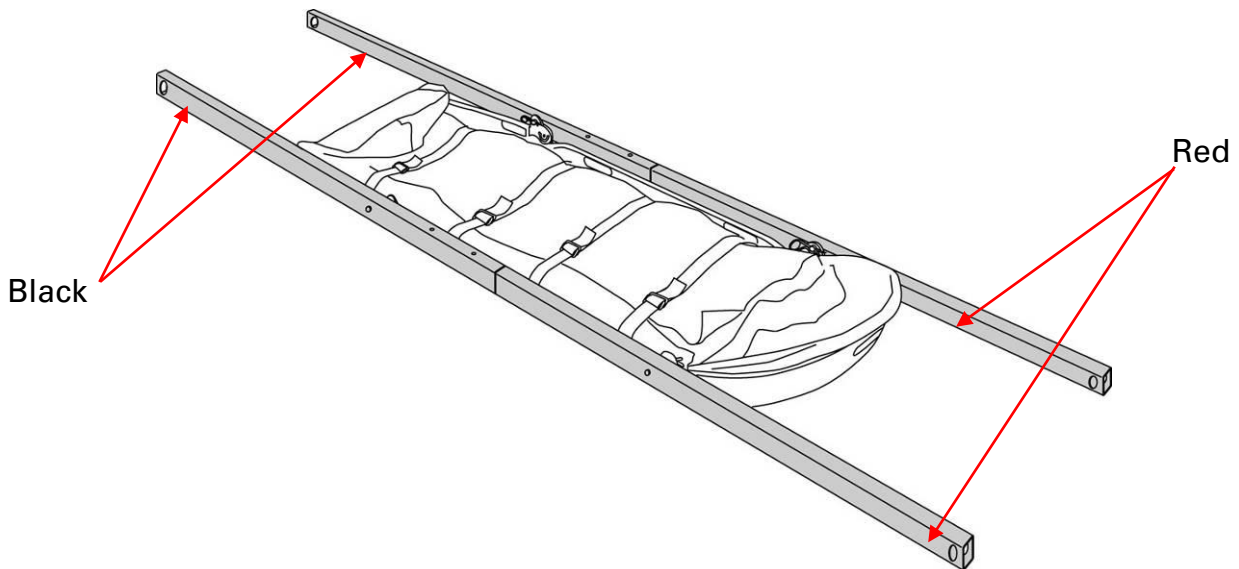

a) Open the plastic loops locking the mounting brackets.

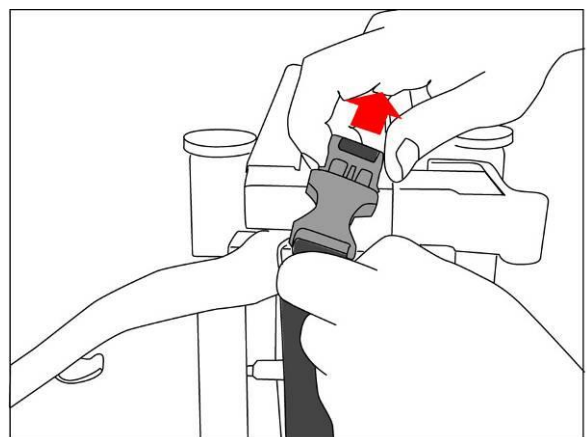



FRANCO GARDA STRETCHER SINGLE WHEEL

User Manual

b) Unfold the mounting brackets and lock them with the snap buttons.



c) Gently lift the stretcher with the patient until the wheel passes under the patient. Then place the stretcher down on the wheel and lock the hooks.





FRANCO GARDA STRETCHER SINGLE WHEEL

User Manual

3/ Use:

When using the wheel the walking direction must be as shown by the sticker.



Walking direction

Caution: Check the tyre pressure before use. (0.5 bar)

4/ Contact:

For any for information or if your equipment requires repair, your only contact is:

TSL RESCUE

9, rue du Pré Faucon
PAE the Glaisins
74940 ANNECY LE VIEUX

Phone: +33 (0)4 50 02 80 29

Fax: +33 (0)4 50 01 28 97

E-mail: info@tslrescue.com

